
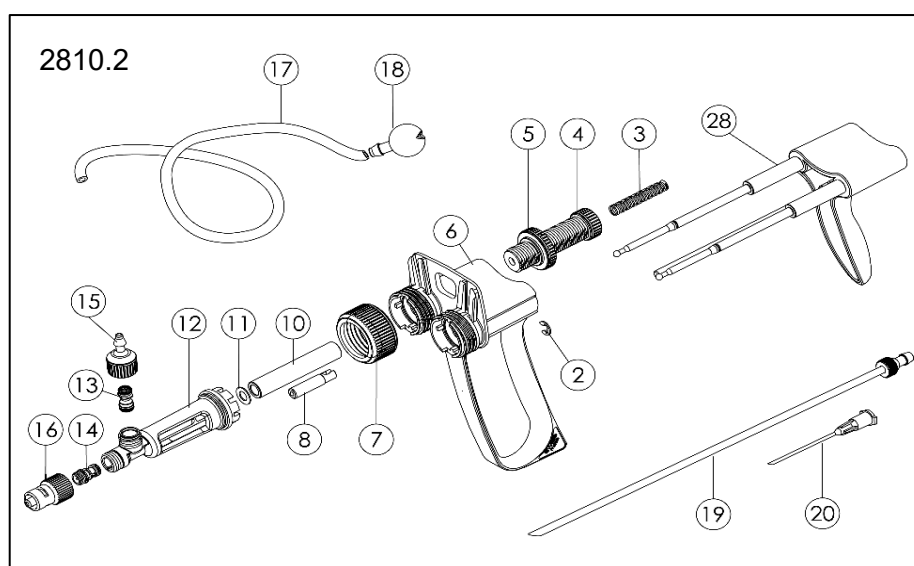


## Spare-parts for Twin syringe *ultra* 2810

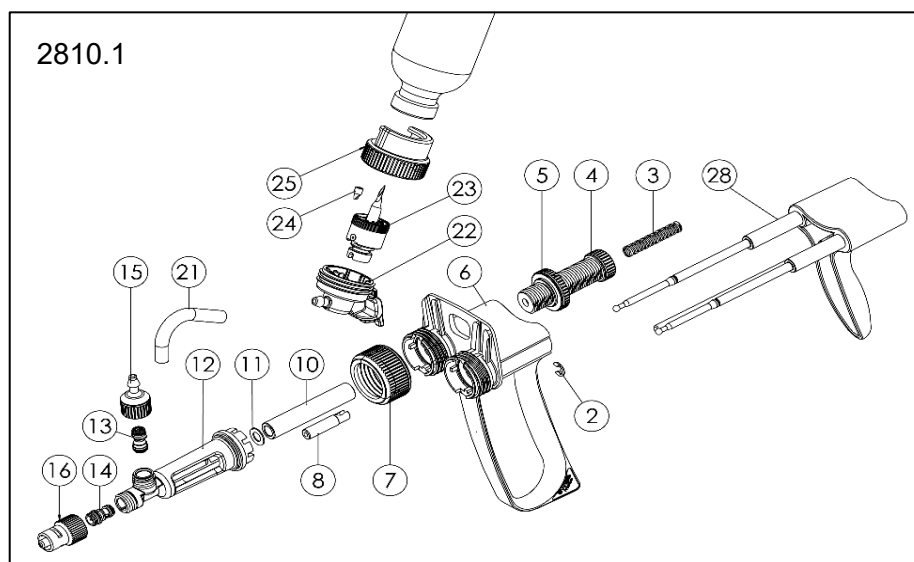

| Draw ref.                                                     | Cat. No.  | Description                       | Comment         |          |
|---------------------------------------------------------------|-----------|-----------------------------------|-----------------|----------|
| Spare parts for self-refilling Twin syringe <i>ultra</i> 2810 |           |                                   |                 |          |
| # 02                                                          | 1810.835  | Clip (10 pces)                    |                 |          |
| # 03                                                          | 1810.961  | Spring, rod (incl. Part #2, clip) |                 | 0.5 mL   |
|                                                               | 1810.962  |                                   |                 | 1 mL     |
|                                                               | 1810.963  |                                   |                 | 2 mL     |
| # 04                                                          | 1810.545  | Socket                            |                 | 0.5 mL   |
|                                                               | 1810.546  |                                   |                 | 1 / 2 mL |
| # 05                                                          | 1810.552  | Nut, socket                       |                 |          |
| # 06                                                          | 2810.1562 | Handle                            |                 |          |
| # 07                                                          | 1810.522  | Nut, casing                       |                 |          |
| # 08                                                          | 1810.585  | Plunger, piston                   | stainless steel | 0.5 mL   |
|                                                               | 1810.586  |                                   |                 | 1 mL     |
|                                                               | 1810.587  |                                   |                 | 2 mL     |
| # 10                                                          | 01810.005 | Barrel (6 pces)                   | Glass           | 0.5 mL   |
|                                                               | 01810.01  |                                   |                 | 1 mL     |
|                                                               | 01810.02  |                                   |                 | 2 mL     |
| # 11                                                          | 1810.925  | Barrel washer (10 pces)*          | FPM             | 0.5 mL   |
|                                                               | 1810.926  |                                   |                 | 1 mL     |
|                                                               | 1810.927  |                                   |                 | 2 mL     |

|      |           |                                                           |                     |              |
|------|-----------|-----------------------------------------------------------|---------------------|--------------|
| # 12 | 1810.1905 | Casing                                                    |                     | 0.5 mL       |
|      | 1810.1906 |                                                           |                     | 1 mL         |
|      | 1810.1907 |                                                           |                     | 2 mL         |
| # 13 | 1810.915  | Cartridge valve, intake*                                  | Red colour          | 0.5 mL       |
|      | 1810.916  |                                                           | Yellow colour       | 1 and 2 mL   |
| # 14 | 1810.945  | Cartridge valve, outlet*                                  | Red colour          | 0.5 mL       |
|      | 1810.946  |                                                           | Yellow colour       | 1 and 2 mL   |
| # 15 | 1810.1635 | Feeding connector                                         |                     | 0.5 mL       |
|      | 1810.1636 |                                                           |                     | 1 and 2 mL   |
| # 16 | 1810.645  | Connector, Luer Lock                                      | Chrome plated brass | 0.5 mL       |
|      | 1810.646  |                                                           |                     | 1 and 2 mL   |
| # 17 | 187.705   | Feed tubing                                               |                     | 1 meter / pk |
|      | 187.710   |                                                           |                     | by the meter |
| # 18 | 187.596   | Sinker                                                    | Chrome plated brass |              |
| # 19 | 187.592   | Feed needle                                               | Chrome plated brass |              |
| # 20 | 187.933   | Vent cannula, plastic (10 pces)                           |                     |              |
| # 21 | 1810.716  | Tubing connector                                          |                     |              |
| # 22 | 1810.371  | Vial holder body                                          |                     |              |
| # 23 | 1810.386  | Needle assembly                                           |                     |              |
| # 24 | 1810.975  | Duckbill (10 pcs)                                         |                     |              |
| # 25 | 1810.374  | Bottle neck                                               |                     |              |
| # 28 | 2810.981  | Hand piece and plunger rod assembly (incl. Part #2, clip) |                     | 0.5 mL       |
|      | 2810.982  |                                                           |                     | 1 / 2 mL     |

\* also available within the spare parts set