


SOCOREX® SELF-REFILLING SYRINGE *ULTRA* 1810


Vial Holder – Quick Guide

Body positioning

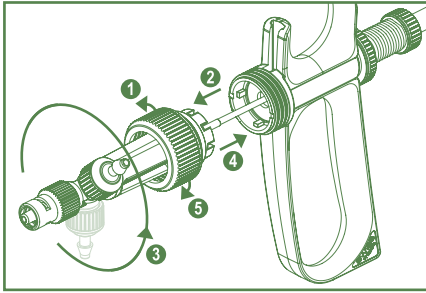


Place intake valve in vertical position.

Assembling

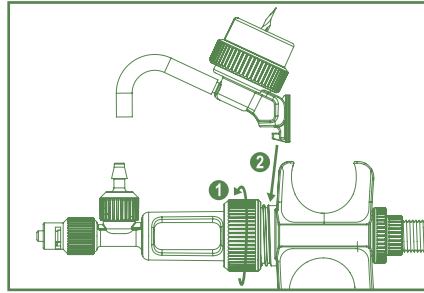


Unscrew syringe nut.
Insert vial holder body until abutment.

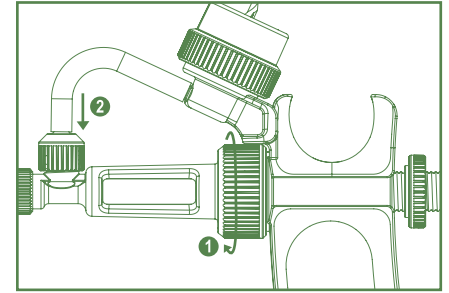


Tighten casing nut to lock vial holder body.
Connect tubing to intake valve.


Bottle mounting

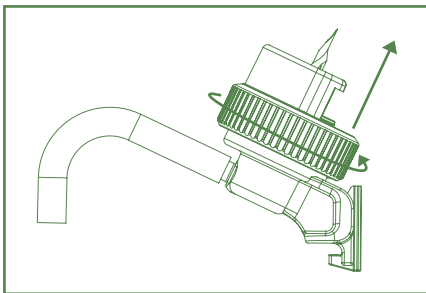

Remove bottle lock.

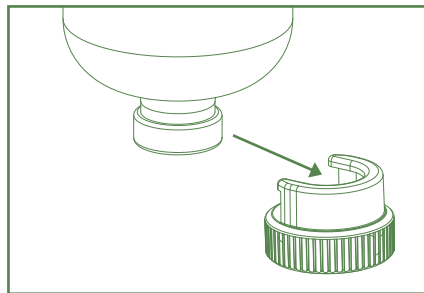


Insert bottle into lock.

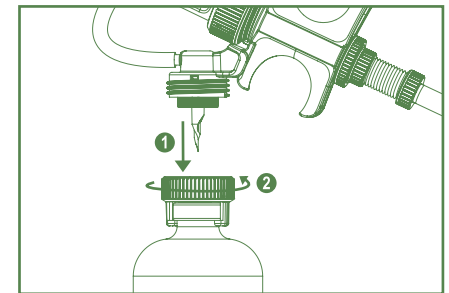

Holding syringe upside down, pierce septum with needle. Screw to tightly lock bottle.


 Do not overtighten (risk of valve cartridge damaged).

Priming

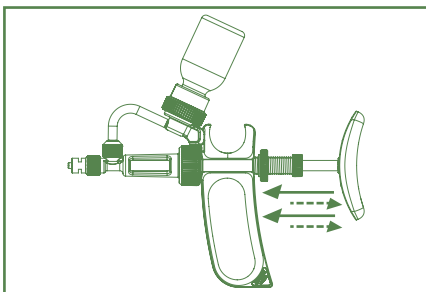


Turn syringe, activate to aspirate and expel liquid.

Bottle removing

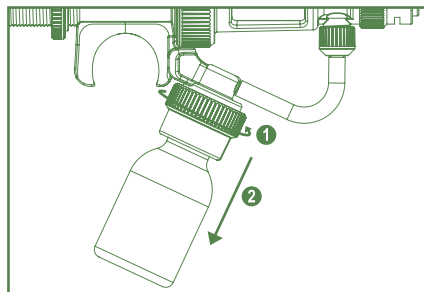


Turn syringe upside down.
Unscrew lock to remove bottle.

Purging and rinsing

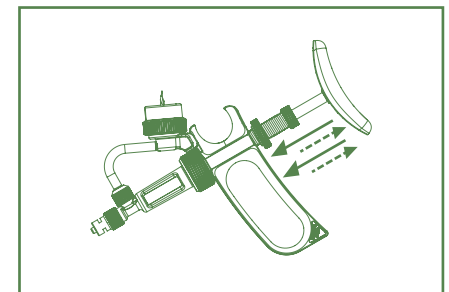


Activate plunger to remove liquid.

Disassembling and cleaning

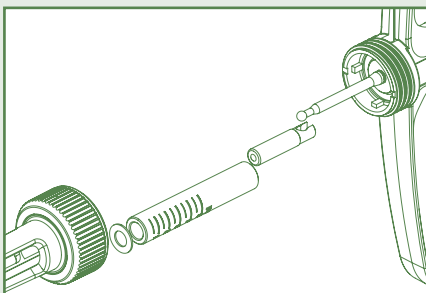


Disassemble and clean according to syringe instructions.

In case of damaged needle

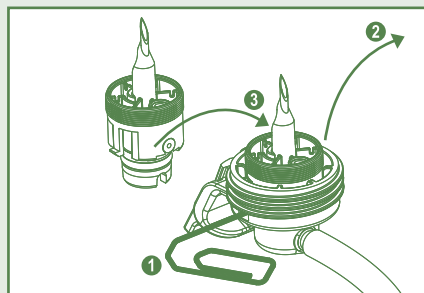

Push clip and pull-out needle assembly.
Insert new needle assembly.

 Sharp needle = risk of injuries
Do not pierce the same bottle several times.

Ordering information

Description	Pack	Cat. No.
Vial holder set	1 / pk	1.1810.010
Spare tubing connector, 55 mm	1 / pk	1810.716
Needle assembly	1 / pk	1810.386
Bottle lock	1 / pk	1810.374



I.1810VIAL.E - A0423