


SOCOREX® SELF-REFILLING SYRINGE *ULTRA* 1810


Quick Guide

Body positioning

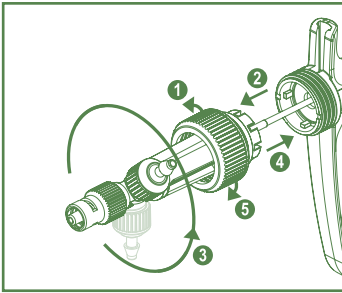


Adjust body position.


Installing tubing

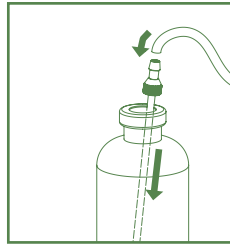

Insert feed needle.
Connect tubing.

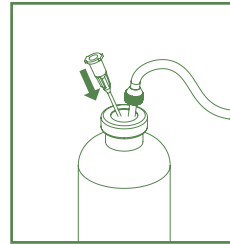


Insert vent cannula.

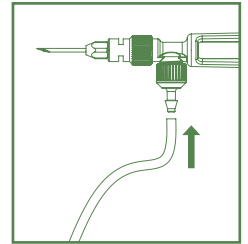


Connect tubing to syringe.

Priming

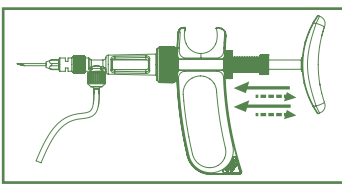


Activate to aspirate and expel liquid.

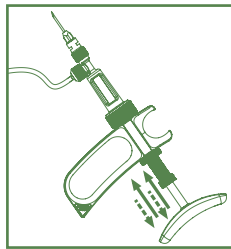


Remove air bubbles.


Volume setting

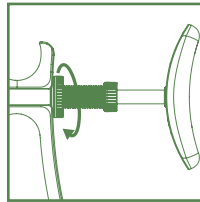

Unlock.

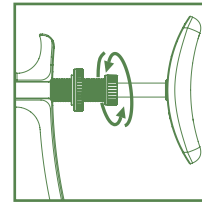


Turn socket to move plunger.

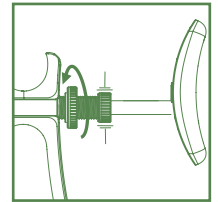


Hold socket and lock.

In case of clogged valve


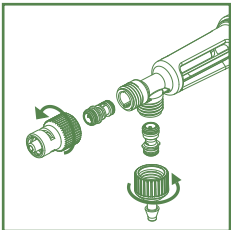


Unscrew and wash valves.
Do not insert any tool
inside valve cartridges.

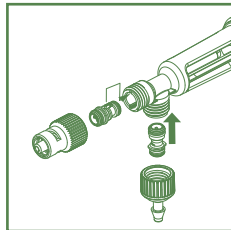


Reassemble according
to mechanical coding.

Volume reading

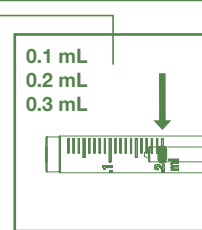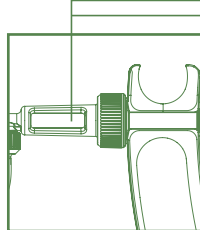


O-ring centered on
graduation.

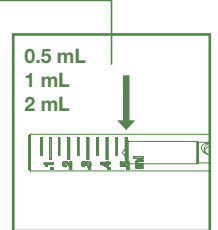


Plunger edge
on graduation.

Purging and rinsing

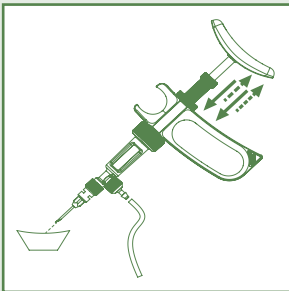

Recover vaccine.
Rinse with water / detergent.


Disassembling and cleaning

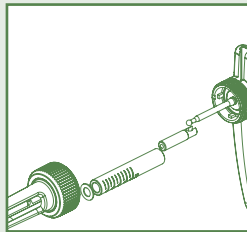


Unscrew nut and separate parts.
Clean and dry parts.



Changing plunger O-ring

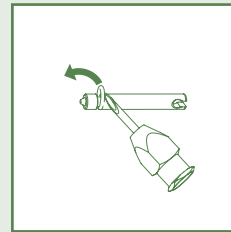


Remove carefully.

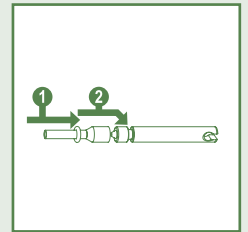


Use guide to roll
new lubricated O-ring
into groove.


Reassembling

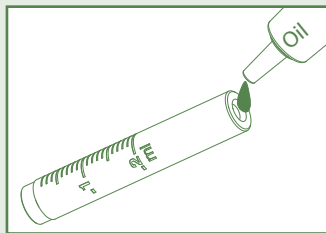

Lubricate barrel entry.

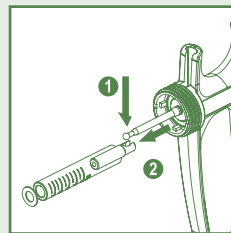

Mount plunger in barrel.
Connect rod.

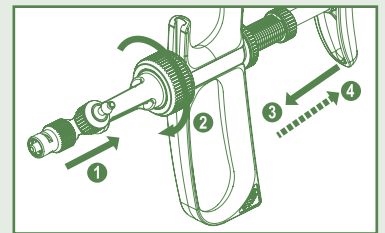

Connect body in pushed position.



I.GUIDE.1810.E - D0223